

CLIP-ONS

NOTES AND INFORMATION

CLIP-ON MARKUS T

– safe handling for maximum visual comfort

1. Insert the clip holder into the gap of the bridge.
2. The gap on all ME Genius frames should be 1.0 mm.
3. The clip holder for clips of the TITAN and ME Genius collections can be shortened to the required length at the notches.



CLIP-ON GREY

Polarised, filter category 3, light absorption approx. 85%, suitable for road traffic, not suitable for night driving.



CLIP-ON BROWN

Back surface AR, filter category 3, light absorption approx. 85%, suitable for traffic, not suitable for night driving.

